Our **veterinary**
product
catalogue.

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50 years
of innovation
through experience

50 years of innovation
through experience

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Our mission is to innovate, develop and commercialize reliable medical devices that guarantee better health care through experience.

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For 50 years we have improved the lives of millions of patients around the world, innovating through experience and prospering together into the future.

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For 50 years, at Internacional Farmaceutica and its Atramat® brand, we have contributed to improving the lives of millions of patients around the world, producing surgical products of the highest quality for the benefit of thousands of surgeons and millions of patients, becoming one of the largest suppliers of surgical sutures in the world, selling to more than 80 countries through a global network of expert business partners.

Our Quality Management System meets the requirements of ISO 13485: 2016; We have the Certifications under the MDSAP scheme (Medical Device Single Audit Program) and Good Manufacturing Practices issued by the Ministries of Health of Mexico (COFEPRIS) and Brazil (ANVISA). In addition, we comply with the standards of numerous regulatory jurisdictions, including the United States of America FDA and the Health Canada. Finally, our products have the CE mark that indicates that they comply with the regulations for medical devices of the European Union and have the approval by the FDA (510k), which allows us to market the Atramat® brand in various markets.

We are a Mexican company whose work ethic, customer service and logistics have allowed a close relationship with our national and international business partners, strengthening the reliability of our products; which, together with the constant training of surgeons and nurses in our extensive portfolio, have made Atramat a brand with high worldwide recognition.

We continue to innovate in our product lines to which our PREMIUM LINE of non-absorbable, absorbable sutures with and without antibacterial coating, knotless wound closure devices; as well as the families of surgical hemostats and minimally invasive devices. In this way, Atramat® confirms that it will continue to provide medical devices that meet the most rigorous quality standards, always with the values and service with which we have been recognized over the years.

I am convinced that this new PREMIUM line will benefit health professionals for an adequate care of their patients and their speedy recovery.

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Atramat
Innovation Through Experience

Suture advice according the type of tissue

Tissue	Absorbable Suture	Non-Absorbable Sutures	Thread Gauge	Type
Skin	• PGC25 • PDX • PGA/PGLA90 • PGLA90 Rapid	• Nylon • Polypropylene	4-0 to 2-0	Reverse Cutting
Subcuticular Closure	• PGC25 • PDX • PGA/PGLA90 • PGLA90 Rapid		4-0 to 2-0	Reverse Cutting
Mucous	• PGC25 • PDX • PGA	• Nylon • Polypropylene	5-0 to 2-0	Reverse Cutting
Intestine	• PGC25 • PDX • PGA	• Nylon • Polypropylene	5-0 to 3-0	Taper Point
Colon	• PGC25 • PDX • PGA	• Nylon • Polypropylene	5-0 to 3-0	Taper Point
Bladder	• PGC25 • PDX • PGA/PGLA90	• Nylon • Polypropylene	5-0 to 3-0	Taper Point
Ureter	• PGC25 • PDX	• Nylon • Polypropylene	5-0 to 4-0	Taper Point
Uterus	• PGC25 • PGA/PGLA90 • PGLA90 Rapid		4-0 to 2-0	Reverse Cutting Taper Point
Ligation of Ovaries	• PGC25 • PDX	• Nylon • Polypropylene • Polyester	3-0 to 0	Reverse Cutting Taper Point
Ligatures	• PGC25 • PDX • PGA/PGLA90	• Nylon • Polypropylene	3-0 to 0	Reverse Cutting Taper Point Without Needle
Muscle	• PGC25 • PDX • PGA/PGLA90	• Nylon • Polypropylene	3-0 to 1	Reverse Cutting Taper Cutting
Abdominal Wall Reconstruction	• PGC25 • PDX • PGA/PGLA90	• Nylon • Polypropylene • Polyester	3-0 to 1	Reverse Cutting Taper Cutting
Hernia	• PDX	• Nylon • Polypropylene • Polyester	3-0 to 0	Reverse Cutting Taper Cutting
Fascia	• PDX	• Nylon • Polypropylene • Polyester	3-0 to 1	Reverse Cutting Taper Cutting
Tendons Ligaments	• PDX • PGA	• Nylon • Polypropylene • Polyester	2-0 to 2	Reverse Cutting Taper Cutting
Articular Capsule	• PDX • PGA/PGLA90	• Nylon • Polypropylene • Polyester	3-0 to 1	Reverse Cutting Taper Cutting
Orthopaedic	• PDX	• Nylon • Polypropylene • Polyester	3-0 to 1	Reverse Cutting Taper Cutting
Blood Vessels	• PGC25 • PDX • PGA/PGLA90	• Nylon • Polypropylene • Polyester	5-0 to 2	Taper Point Taper Cutting
Nerves	• PDX	• Nylon • Polypropylene • Polyester	6-0 to 2-0	Taper Point Taper Cutting
Eyelid	• PGC25 • PDX • PGA/PGLA90 • PGLA90 Rapid	• Nylon • Polypropylene	6-0 to 3-0	Triangular Cilindrica
Eye	• PGA/PGLA90	• Nylon • Polypropylene	6-0 to 3-0	Reverse Cutting Taper Point
Cornea	• PGA/PGLA90	• Nylon	10-0 to 7-0	Spatula
Dental	• PGC25 • PDX • PGA/PGLA90 • PGLA90 Rapid	• Nylon • Polypropylene • Silk	5-0 to 2-0	Reverse Cutting

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Atramat® PGC25 Surgical Suture

General Characteristics

PGC25 Atramat® is a monofilament, synthetic absorbable copolymer of glycolide and caprolactone [poly(glycolide-co-epsilon-caprolactone)].

Characteristics

- High tensile strength.
- PGC25 Atramat® is indicated in surgeries for the closure of surgical layers, which requires a suture material with a useful resistance period of approximately 14-18 days.
- Absorption by hydrolysis.
- Monofilament, uniform diameter.
- Violet or colorless.
- Sterilization by Ethylene Oxide.
- Expiration 3 years.

Absorption

- Loss of tensile strength: 50% at 14 days, 0% at 28 days.
- Absorption is produced by progressive hydrolysis and is complete after 110 days.

Benefits

- Plastic knot coupling providing excellent knot stability.
- Perfect adaptation to the edges of the wound without altering its irrigation due to its good flexibility.
- Elastic and very easy to handle.
- Easy recovery from memory effect
- No capillarity.
- Excellent visibility in the surgical field.
- High resistance to:
 - Traction.
 - Knot breakage.
- Adequate tissue support.
- Predictable absorption rate.
- Smooth passage through tissue.

Applications

It is indicated for:

- The soft approach.
- Suturing limbs, anatomical structures with cavities and hollow liquid-filled organs.
- General and digestive surgery.
- Gynecology and obstetrics.
- Urology.
- Plastic surgery.
- Ligatures.

Knotting Recommendations

It is recommended to use thick gauges (2 - 1 - 0 - 2/0) USP. Make a surgeon's knot, plus a simple knot. This allows the tension to be adjusted. Once adjusted, it is advisable to add a square knot.

In fine gauges (3/0 - 4/0 - 5/0 - 6/0) USP, two initial square knots are recommended with two or three additional fixing knots leaving the ends 4 or 5 mm in length, depending on the case and the preference of the surgeon.

High Quality Needles

Excellent quality 300 grade stainless steel needles with an optimum geometric design and a technologically innovative coating, resulting in an ideal needle.

ATRAGLIDE Needles: A combination of geometry, sharpness and technology with outstanding performance facilitating penetration and smooth passage through tissue.

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Absorbable Sutures
Suturas Absorbibles
Sutures Résorbables

Atramat® PGC25

Poly(glycolide-co-epsilon-caprolactone)

Taper Point
Cilindrica
Ronde

1/2 Circle
Círculo
Cercle

Thread Gauge
Calibre de la Hebra
Calibre du Brin

Type Tipo Type	Needle Aguja Aiguille	Gauge Hebra Brin	USP 6/0	USP 5/0	USP 4/0	USP 3/0	USP 2/0	USP 0	USP 1
SR13 13 mm		70 cm	SR1306	SR1305					
SR17 17 mm		70 cm	SR1706	SR1705	SR1704	SR1703			
SR20 20 mm		70 cm			SR2004	SR2003			
SR26 26 mm		70 cm			SR2604	SR2603	SR2602	SR2601	
SR30 30 mm		90 cm					SR3002-90	SR3001-90	
SG36 36 mm		90 cm					SG3602-90	SG3601-90	
SG40 40 mm		90 cm							SG4008-90

Reverse Cutting
Triangular
Triangulaire

1/2 Circle
Círculo
Cercle

Thread Gauge
Calibre de la Hebra
Calibre du Brin

Type Tipo Type	Needle Aguja Aiguille	Gauge Hebra Brin	USP 4/0	USP 3/0	USP 2/0	USP 0	USP 1	USP 2
P20 20 mm		70 cm	P2004					
P22 22 mm		70 cm		P2203				
P26 26 mm		70 cm 90 cm	P2604 P2604-90	P2603 P2603-90	P2602 P2602-90	P2601 P2601-90	P2608 P2608-90	P2609-90
P30 30 mm		70 cm 90 cm			P3002 P3002-90	P3001 P3001-90		
P36 36 mm		90 cm			P3602-90	P3601-90	P3608-90	P3609-90
P40 40 mm		90 cm					P4008-90	

Reverse Cutting
Triangular
Triangulaire

3/8 Circle
Círculo
Cercle

Thread Gauge
Calibre de la Hebra
Calibre du Brin

Type Tipo Type	Needle Aguja Aiguille	Gauge Hebra Brin	USP 6/0 USP 5/0 USP 4/0 USP 3/0 USP 2/0 USP 0 USP 1							
PE13 13 mm		45 cm	PE1306-45							
PE16 16 mm		45 cm	PE1606-45	PE1605	PE1604	PE1603				
		70 cm								
PE19 19 mm		70 cm			PE1904	PE1903				
PE24 24 mm		70 cm				PE2403	PE2402	PE2401	PE2408	
		90 cm				PE2403-90	PE2402-90	PE2401-90	PE2408-90	
PE30 30 mm		70 cm						PE3001		
		90 cm						PE3001-90		
PE36 36 mm		90 cm					PE3602-90	PE3601-90		
PE40 40 mm		90 cm							PE4008-90	

Atraglide®
Atraglide®
Atraglide®

3/8 Circle
Círculo
Cercle

Thread Gauge
Calibre de la Hebra
Calibre du Brin

Type Tipo Type	Needle Aguja Aiguille	Gauge Hebra Brin	USP 6/0 USP 5/0 USP 4/0 USP 3/0			
HPE13 13 mm		45 cm	HPE1306-45B			
HPE16 16 mm		45 cm		HPE1605-45	HPE1604-45B	HPE1603-45B

Straight Cutting
Recta
Droite

3/8 Circle
Círculo
Cercle

Thread Gauge
Calibre de la Hebra
Calibre du Brin

Type Tipo Type	Needle Aguja Aiguille	Gauge Hebra Brin	USP 3/0 USP 2/0	
PU60 60 mm		70 cm	PU6003-70	PU6002-70

Tapercut
Trocar
Trocar

1/2 Circle
Círculo
Cercle

Thread Gauge
Calibre de la Hebra
Calibre du Brin

Type Tipo Type	Needle Aguja Aiguille	Gauge Hebra Brin	USP 4/0 USP 3/0 USP 2/0 USP 0 USP 1					
PW17 17 mm		70 cm	PW1704					
PW26 26 mm		70 cm 90 cm	PW2604	PW2603	PW2602	PW2601		PW2608-90
PW36 36 mm		90 cm			PW3602-90	PW3601-90		PW3608-90

Absorbable Sutures

Atramat® PDX Surgical Suture

General Characteristics

The PDX Atramat® absorbable surgical suture is manufactured from p-dioxanone monomer, a monofilament, synthetic material absorbed by hydrolysis.

Characteristics

- Long-term absorbable synthetic suture.
- High tensile strength.
- Absorption by hydrolysis.
- Monofilament with uniform diameter.
- Violet color.
- Sterilization by Ethylene Oxide.
- Expiration 5 years.

Absorption

- Loss of tensile strength: 70% at the fourth week and 50% at the sixth week.
- Absorption is produced by progressive hydrolysis and is complete after 180-220 days.

Benefits

- Ensures safe wound support without interfering in the critical healing process.
- It is frequently used in the coaptation of tissues that require a long healing period.
- Easy recovery from memory effect.
- Surgeries in patients with delayed healing, immunosuppressive, diabetes, advanced age or obesity.
- Adapts perfectly to the edges of the wound without altering its irrigation, thanks to its good flexibility.
- Plastic knot coupling providing total safety.
- Absence of capillarity.
- Excellent visibility in the surgical field.
- High resistance to:
 - Traction.
 - Knot breakage.
- Smooth passage through tissue.

Applications

PDX Atramat® is indicated in the general approach of soft tissues, especially when a prolonged support of the wound edges, more than 4 weeks.

- Fascia and sternum closure.
- General Surgery.
- Digestive surgery.
- Traumatology and orthopedics (tendons, ligaments, nerves, small vessels).
- Pediatric surgery of cardiovascular tissues (anastomosis of veins and arteries).
- Vascular surgery (peripheral vessels).

Knotting Recommendations

In thick gauges (2, 1, 0, 2-0). U.S.P. it is recommended to make a surgeon's knot plus a simple knot to enable the surgeon to adjust the tension. Once adjusted, it is advisable to add a square knot depending on the case and the experience of the surgeon.

In thin calibers (3-0, 4-0, 5-0, 6-0, 7-0) USP, two square knots are initially recommended with two or three additional fixing knots leaving the ends 4 or 5 mm in length, depending on the case and the preference of the surgeon.

High Quality Needles

Excellent quality 300 grade stainless steel needles with an optimum geometric design and a technologically innovative coating, resulting in the ideal needle.

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Atramat® PGLA90 Surgical Suture

General Characteristics

The PGLA90 Atramat® surgical suture is composed of a copolymer of glycolic acid and L-lactide [poly (glycolide-co-L-lactide)], braided, sterile and absorbed by hydrolysis. The strands have a coating of poly (glycolide-co-L-lactide) and calcium stearate.

Characteristics

- High tensile strength.
- Cylindrical braided multifilament with coating.
- Absorption by hydrolysis.
- Violet or beige (undyed).
- Sterilization by Ethylene Oxide.
- Expiration 5 years.

Absorption

- Loss of tensile strength; 75% by the second week.
- The absorption is produced by progressive hydrolysis and is complete after 60-80 days.

Benefits

- PGLA90 Atramat® offers a predictable absorption rate and high tensile strength, above the USP standard, providing artificial tissue support during the critical healing process.
- PGLA90 Atramat® in its violet form allows for perfect visibility in the surgical field and easy identification as an absorbable synthetic suture. It is also available in undyed form, indicated in intradermal superficial suture.
 - Excellent knot coupling.
 - Easy of handling.
 - High resistance to:
 - Traction.
 - Knot breakage.
 - Smooth passage through the tissues.

Applications

Used in coaptation of soft tissues, ligatures, and in a wide variety of surgical specialties.

- General Surgery.
- Digestive surgery.
- Subcutaneous.
- Gynecology and obstetrics.
- Traumatology.
- Urology.
- Ophthalmic Surgery.
- Plastic surgery.
- Minimally invasive surgery (M.I.S.).
- Ligatures.

Knotting Recommendations

As with most braided suture materials, it is recommended to make a square knot with two or three additional fixing knots leaving the ends 2 or 3 mm in length, depending on the case and the preference of the surgeon.

High Quality Needles

Excellent quality in its 300 series stainless steel needles with an optimum geometric design and a technologically innovative coating, thus achieving an ideal needle.

ATRAGLIDE Needles: Geometry, sharpening and technology with outstanding performance facilitating penetration and smooth passage through the tissues.

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Atramat® PGA Surgical Suture

General Characteristics

The Atramat® PGA surgical suture is composed of homopolymer filaments of glycolic acid, braided, sterile and absorbed by hydrolysis. The strands have a coating of polycaprolactone and calcium stearate.

Characteristics

- High tensile strength.
- Cylindrical braided multifilament with coating.
- Absorption by hydrolysis.
- Violet or Undyed.
- Sterilization of Ethylene Oxide.
- Expiration 5 years.

Absorption

- Loss of tensile strength: 70% at the second week and 50% at the third week.
- Absorption is produced by the progressive hydrolysis and is complete after 60-90 days.

Benefits

- PGA Atramat® offers a predictable absorption rate and high tensile strength, above the USP standard, providing artificial tissue support during the critical healing period.
- PGA Atramat® in its violet form allows for perfect visibility in the surgical field and easy identification as an absorbable synthetic suture. Also available in undyed form, indicated in intradermal superficial suture.
 - Excellent knot coupling.
 - Superior handling.
 - High resistance to:
 - Traction.
 - Knot breakage.
 - Smooth passage through tissue.

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High Quality Needles

Excellent quality 300 grade stainless steel with an optimum geometric design and a technologically innovative coating, resulting in the ideal needle.

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Absorbable Sutures

Atramat® PGLA90 Rapid Surgical Suture

General Characteristics

The surgical suture of rapid absorption PGLA Rapid Atramat® is composed of copolymer of glycolic acid and L-lactide [poly (glycolide-co-L-lactide)], braided, sterile and absorbed means of a hydrolysis process. The strand contains a coating of poly [glycolide-co-L-lactide) and Calcium Stearate.

Characteristics

- High tensile strength.
- Cylindrical braided multifilament with coating.
- Absorption by hydrolysis.
- Undyed.
- Sterilization of gamma radiation.
- Expiration 5 years.

Absorption

- Loss of tensile strength; 50% at the end of the first week.
- Absorption occurs by progressive hydrolysis in approximately 42 days.

Benefits

PGLA90 Rapid Atramat® offers a predictable absorption rate and high tensile strength, above the USP standard, providing artificial tissue support during the critical healing period.

- Excellent knot coupling.
- Ease of superior management.
- High resistance to:
 - Traction.
 - Knot breakage.
- Soft step through the tissues.

Applications

PGLA90 Rapid Atramat® is indicated for the closure of rapidly healing layers in all surgical specialties.

- General Surgery.
- Subcutaneous cell plan.

- Intradermal suture.
- Gynecology and Obstetrics. Vaginal mucous closure, episiotomy, treatment of urinary incontinence.
- Urology.
- Ophthalmic Surgery (closure of conjunctiva).
- Plastic surgery.
- Ligatures.

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High Quality Needles

Excellent quality 300 grade series stainless steel with an optimum geometric design and a technologically innovative coating, resulting in the ideal needle.

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Atramat® Nylon Surgical Suture

General Characteristics

The Nylon Atramat® surgical suture is a cylindrical monofilament, synthetic, non-absorbable, sterile suture, composed of long chain aliphatic polymers Nylon 6 and Nylon 6.6.

Characteristics

- Synthetic suture, non-absorbable, monofilament, black.
- Nylon is gradually encapsulated by fibrous connective tissue.
- Excellent knot coupling.
- Monofilament with uniform thickness.
- Sterilization by Ethylene Oxide.
- Expiration 5 years.

Absorption

- Non-absorbable suture.

Benefits

- Plastic knot coupling providing excellent knot security.
- Its tensile strength and controlled elongation ensure the correct approximation of the tissue edges.
- Elastic and easy to handle.
- Absence of capillarity.
- Excellent visibility in the surgical field.
- Smooth passage through tissue, guaranteeing perfect healing.
- Nylon Atramat® uses a wide range of cylindrical needles with precision sharpening allowing a very favorable cosmetic result.

Applications

Its physical characteristics in addition to minimal tissue reactivity make it the ideal suture for a wide variety of tissues.

- General Surgery.
- Gynecology and Obstetrics.

- Traumatology.
- Plastic and reconstructive surgery.
- Micro-vascular surgery.
- Ophthalmic Surgery.

Knotting Recommendations

In thick gauges (2 - 1 - 0 - 2/0) USP it is recommended to make a surgeon's knot plus a simple knot to facilitate tension adjustment. Once adjusted, it is advisable to add a square knot.

In fine gauges (3/0 - 4/0 - 5/0 - 6/0) USP, two square knots are initially recommended with two or three additional fixing knots leaving the ends 4 or 5 mm in length, depending on the case and the preference of the surgeon.

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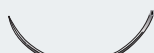
Non Absorbable Sutures

Atramat® Polypropylene Surgical Suture

General Characteristics

The Atramat® Polypropylene surgical suture is a sterile, non-absorbable, synthetic monofilament suture made from an isotactic stereoisomer of the polypropylene homopolymer.

Characteristics

- Synthetic suture, non-absorbable, monofilament, blue in color.
- Excellent histocompatibility and made of the best quality raw materials.
- Monofilament with a very uniform cylindrical diameter.
- Synthetic and inert material with minimal tissue reaction.
- Premium needles with precision sharpening.
- Support with needle guard.
- Sterilization by Ethylene Oxide.

Absorption

- Non-absorbable suture.

Benefits

- Plastic knot coupling providing excellent security.
- Resistance to repeated bending.
- Excellent plastic-deformation.
- Adequate long-term tissue support.
- Tensile strength and controlled elongation ensure correct approximation of tissue edges.
- Ideal biocompatibility.
- Ideal in cardiovascular procedures including cardiac prosthesis and vascular anastomosis, where the beating of the heart has to be withstood.
- Absence of capillarity.
- Excellent visibility in the surgical field.
- Smooth passage through tissue, avoiding tissue drag and guaranteeing perfect healing.

Applications

Its physical characteristics, together with its minimal tissue reactivity, make it the ideal suture for a wide variety of tissues.

- General Surgery.
- Gynecology and obstetrics
- Traumatology.
- Plastic and reconstructive surgery.
- Cuticle.
- Ophthalmic Surgery.
- Plastic and Reconstructive Surgery

Cardiovascular Surgery - Micro-Vascular Surgery Easy Penetration Without Filtration

The Atramat® Polypropylene suture is made with precision sharpened cylindrical needles specially designed to give a favorable cosmetic result. In addition, our needles are superior in terms of bending, strength and hardness.

The Rockwell hardness of the needles used in Atramat® Polypropylene sutures is higher than that of conventional needles, allowing for excellent penetration facilitating passage through calcified tissues or atheromatous plaques.

ATRAMAT® has special needles for cardiovascular use, providing easy penetration into tissue, and characterized by a uniform needle-thread diameter. This affords perfect balance and helps prevent filtration, making it ideal for vascular use.

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Non Absorbable Sutures

Atramat® Polyester Surgical Suture

General Characteristics

The Atramat® Polyester suture is a sterile, non-absorbable braided multifilament suture made of high-molecular-weight, long-chain Polyester fiber, with aromatic rings made of terephthalic acid or its esters and ethylene glycol.

Characteristics

- Synthetic, non-absorbable, braided multifilament suture, green in color (ophthalmic white).
- Resistance to repeated bending.
- Excellent histocompatibility and made of the best quality raw materials.
- Synthetic and inert with minimal tissue reaction.
- Sterilization by Gamma Ray or Ethylene Oxide.
- Expiration 5 years.

Absorption

- Non-absorbable suture.

Benefits

- Precise positioning and excellent knot security.
- Excellent handling.
- Available in single and multipack containers.
- Its high tensile strength ensures the correct approximation of tissue edges.
- Knot resistance is retained indefinitely.
- Good long-term tissue support.
- Ideal for use in Cardiovascular Surgery (cardiac prostheses).
- Excellent visibility in the surgical field.
- Smooth passage through tissue, guaranteeing perfect healing.
- Excellent tissue penetration with minimal tissue trauma.
- Excellent biocompatibility.
- Available with Teflon (PTFE) pledgets, ideal in cardiovascular surgery.

Applications

Frequently used in a variety of surgical specialties. Its physical characteristics make it the ideal suture for a wide variety of tissues.

- Cardiac and Vascular Surgery.
- General Surgery.
- Thoracic surgery.
- Traumatology.
- Plastic surgery
- Ophthalmic Surgery.

High Quality Needles

Atramat® has developed a new generation of needles which perfectly combine a special 300 grade stainless steel alloy with an optimum geometric design and a technologically innovative coating, resulting in the perfect needle.

Supraglyde needles provide safety and control inside tissue, facilitating penetration into hard tissues (calcified and / or atheromatous plaques) and reducing filtration.

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Atramat® Silk Surgical Suture

General Characteristics

The Silk Atramat® surgical suture is a sterile, non-absorbable multifilament suture made from an organic protein, fibroin, obtained from the Bombyx Mori silkworm from the Bombycidae family.

Characteristics

- Made from the highest quality medical grade raw material.
- High tensile strength.
- Cylindrical braided multifilament with silicone coating.
- Needle guard (where applicable).
- Black or blue, with or without silicone coating.
- Also available in white (virgin silk) for ophthalmology, without silicone coating.
- Sterilization by Ethylene Oxide.
- Expiration 5 years.

Absorption

- Non-absorbable suture.

Benefits

- Guaranteed premium quality silk.
- Excellent handling.
- The advanced technology used in the braiding process ensures a compact thread with high tensile strength.
- Increased tissue support time.
- Its silicone coating allows for smooth passage through tissue, avoiding any saw effect and resulting in a non-capillary thread.
- Good visibility in the surgical field.
- Versatile.

Applications

Used in the coaptation of soft tissues, ligatures, and in a wide variety of surgical specialties.

- General Surgery.
- Closing the skin.
- Oral surgery.
- Neurosurgery.
- Ophthalmic surgery (virgin silk).
- Ligatures.

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Atramat® **Plus**

Antibacterial Surgical Suture

• Atramat® Plus is a reabsorbable suture coated with chlorhexidine diacetate, a composition that inhibits the growth of gram-positive and gram-negative bacteria and acts as a fungicide against yeast.

• Our antibacterial sutures have been shown in vitro and in experimental models to be effective in preventing bacterial colonization in the suture itself. Its coating with chlorhexidine generates an inhibition around it to reduce the risk of infections, thus eliminating the probability of infection in the incision and preventing a possible postoperative infection.

• Atramat Plus has proven effect against:

1. Staphylococcus aureus
2. Staphylococcus epidermidis
3. Escherichia coli

Chlorhexidine diacetate antibacterial coating increases wound closure effect, providing more safety for the patient.

Our antibacterial sutures maintain all the features and benefits expected of any Atramat® suture.

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Atrammat PROPYMESH

General Characteristics

A surgical mesh of monofilament of non-absorbable polypropylene used to repair defects of the abdominal wall. The ideal choice for optimal surgical results.

- The special structure of the mesh prevents fraying when cut.
- It is indicated for both minimally invasive surgery and open surgery.

Atrammat® PROPYMESH has an interwoven homogeneous structure that provides controlled elongation in two directions, providing adequate elasticity thereby favoring the anatomical movement of the abdominal wall.

Material

Surgical mesh made of monofilament polypropylene.

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Benefits

Monofilament: decreases the probability of infection.

Bidirectional elasticity: Adapts to the anatomical movements of the abdominal wall.

Mesh structure: Allows the colonization of cells favoring the growth of blood vessels around the mesh.

Adequate pore size: rapid and uniform colonization, minimal risk of seroma formation.

Without memory: excellent handling.

The edges: laser-cut, avoiding fraying.

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Atramat PROPYMESH Flex

General Characteristics

The partially absorbable, synthetic and sterile surgical mesh, Atramat PROPYMESH Flex® is a mesh made in equal parts of non-absorbable polypropylene monofilament and Poly Glycolide-co-Epsilon-caprolactone (PGC25) which gives it its characteristic of being partially absorbable. Once the Poly Glycolide-co-Epsilon-caprolactone (PGC25) has been absorbed, only the polypropylene mesh remains. The structure and size of this remaining mesh are designed to withstand the physiological stresses to which the abdominal wall is subjected. The substances contained in the mesh are non-collagens, non-allergenic, non-antigenic, non-pyrogenic and non-toxic. The Atramat PROPYMESH Flex® surgical mesh is composed of stained parts for identification.

Surgical technique

The Atramat PROPYMESH® polypropylene monofilament surgical mesh is implanted according to currently accepted procedures for surgical mesh placement, either in open surgery (Lichtenstein, TIPP) or Laparoscopic (TAPP, TEP). Adequate preparation and sufficient mesh size should be considered to avoid the possibility of risk of insufficient coverage of the injury. After ensuring that the mesh is placed without tension and free of cavities and folds, it must be held in place according to the implantation technique either with sutures or clips.

Uses:

- Perineal hernia.
- Inguinal hernia.
- Incisional hernia.
- Abdominal hernia.
- Abdominal wall reconstruction.

How it works

The Atramat PROPYMESH Flex® surgical mesh can cause a minimal inflammatory reaction in tissues, which is followed by gradual encapsulation by fibrous connective tissue. The gradual loss of tensile strength and the absorption of mesh occurs through a hydrolysis process, where the poly (glycolide-co-epsilon-caprolactone) is absorbed and metabolized by the organism. Absorption is initiated by the loss of tensile strength, followed by the loss of mass. The absorption of PGC25 ended between 90 and 110 days, leaving only the Polypropylene portion, offering greater resistance with the least amount of foreign body, to the patients greater comfort and mobility.

Expiration

5 years from the date indicated on the label.

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Recommended suture for mesh fixation.

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Curaspon®

General Characteristics

- Haemostatic made of pharmaceutical gelatin highly purified, neutral PH.
- Absorbs up to 50 times of its own weight and its easily attached.
- Quick and full absorption.
- Recommended for brain and neck surgeries.
- Can be use in endoscopic surgery.
- Without risk of allergic reaction.
- Available in powder form to mix it with serum and create the gelatin.
- Can be cut.
- Anti-adhesives on surgical instruments.
- Double sterile packaging presentation.

Principle of action

Curaspon® has a prosity structucte that activates the thrombocytes at the moment blood comes in contact with the matrix of the sponge. This causes the thrombocytes to release a series of substances which promote their aggregation at the same time as their surfaces change character, thus enabling them to act as a catalyst for the formation of the fibrin.

Application and absorption

Curaspon® makes the haemostasis in 2-4 minutes. With that we are able to control the bleeding.
Fixed completely to the tissue surface.
Can be used dry or impregnated with saline solution.
Liquefies in 2 - 5 days in contact with mucosa.
Completely absorbed in 3-4 weeks.

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CuraCel® Standard

General Characteristics

- Plant-based haemostat, avoids the possibility of exposure to contaminants of animal or human origin.
- Suitable for endoscopic procedures.
- Can be cut without fraying.
- Can be used in cavities (after having remove a tumor).
- Bactericide.
- High flexibility .

Principle of action

La Curacel® range do the haemostasis due to its low pH, causing the denaturing of the albumin and globuline. The saturated gel supports the formation of blood clots.

Application and absorption

CuraCel® Standard makes the haemostasis in 3-4 minutes.
Ready-to-use dry.
Can be placed in, hold or wind the bleeding.
Completely absorbed within 7-14 days.

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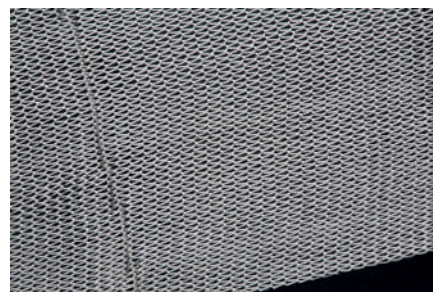
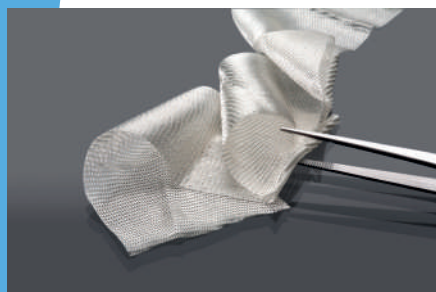
CuraCel® High Density

General Characteristics

- Plant-based haemostat, avoids the possibility of exposure to contaminants of animal or human origin.
- Thick and dense tissue to control heavier bleedings.
- Provides greater resistance. Can be used as supporting tissue in the suture of friable tissues.
- Flexible, adaptable and resistant.
- Density suitable for suturing.
- Does not stick or fall apart.
- Can be used via endoscopic.
- Bactericide.
- Store at room temperature.

Principle of action

La Curacel® range do the haemostasis due to its low pH, causing the denaturing of the albumin and globuline. The saturated gel supports the formation of blood clots.

Application and absorption

CuraCel® makes the haemostasis in 3-4 minutes.
Ready-to-use dry.
Can be placed in, hold or wind the bleeding.
Completely absorbed within 7-14 days.

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CuraCel® Fibrilar

General Characteristics

- Plant-based haemostat, avoids the possibility of exposure to contaminants of animal or human origin.
- 7-layer fibrillose fabrics.
- Can be apply in layers, increasing the surface to be treated.
- Can be separated in small pieces without cutting it. This facilitate the application of the correct dose.
- Suitable for laparoscopic procedures.
- Bactericide.

Principle of action

La Curacel® range do the haemostasis due to its low pH, causing the denaturing of the albumin and globuline. The saturated gel supports the formation of blood clots.

Application and absorption

CuraCel® makes the haemostasis in 3-4 minutes.
Ready-to-use dry.
Can be placed in, hold or wind the bleeding.
Completely absorbed within 7-14 days.

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CuraTamp®

General Characteristics

- Sterile, absorbable haemostat made with oxidised cellulose gauze.
- Can be cut without fraying.
- Huge flexibility.
- Suitable for laparoscopic procedures.
- No fabric memory effect.
- Bactericide.

Principle of action

Oxidised Cellulose Curatamp® is a fabric material that is obtained by the oxidation of alpha grade cotton gauze using nitrous oxide.

The low pH of the cellulosic acid within the product has caustic properties that lead to haemostasis via the initial denaturation of blood proteins.

Application and absorption

CuraTamp® makes the haemostasis in 3-4 minutes.
Ready-to-use dry.
Completely absorbed within 7 - 14 days.

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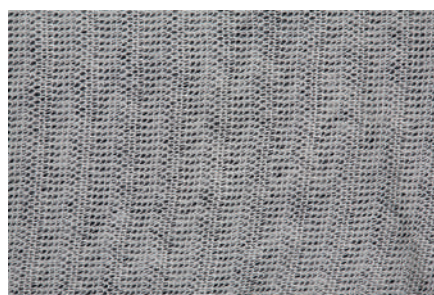
TenaTac®

TenaTac® is an haemostatic absorbable gelatine sponge with Selentus technology (unique surface modified with laser in order to improve the tissue adhesion).

The modification is done with high precision. The flexible columns conform to the variations in tissue surface and act independently to resist shear forces that may dislodge a normal sponge in the peri-operative period.

General Characteristics

- Laser-guided surface modification ensures that it adheres strongly to the tissue.
- Controls bleeding rapidly and seals off bleeding sites.
- Reduced risk of migration from site of application.
- Ready-to-use dry.
- Free from blood-derived proteins.
- Easy to apply.

Application and absorption

TenaTac® makes the haemostasis in 3 minutes.
Liquefies in 2 - 5 days in contact with mucosa.
Completely absorbed in 4-6 weeks.

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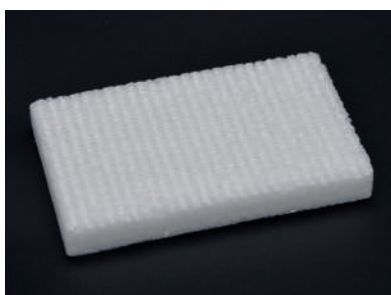
Atramat® Bonewax

General Characteristics

Bonewax Atramat® is composed of a mixture of bee wax obtained from the walls of the honeycomb of the *Apis mellifera* Linne and Isopropyl Palmitate, which has a solvent exhibiting good emulsion. It has an opac appearance and has a characteristic colour.

Principle of action

Once the Atramat Bone Wax is opened, it must be softened using an aseptic technique, until the desired consistency is achieved, which will be applied to the bone surface as required in the surgical procedure and the surgeon's preference.

Atramat Bone Wax achieves local haemostasis of bones by acting as a mechanical barrier (plug), as it is a non-absorbable and inert material that does not have a biochemical action in the body.

It is only recommended as a local haemostatic for bone.

Code	Description	Size	Packaging (units)
CH-2.5N	Bonewax	2.5 gr.	12

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Mfg: xxxxxxxxxxxx
Exp: xxxxxxxxxxxx

STERILE R
LOT XXXXXXXX

Atramat
Innovate in medical devices

Bone Wax / Cera para Huesos
060.196.0057

Suggested Use: Surgical Hemostatic
Uso Sugerido: Hemostático Quirúrgico
Non toxic / Non Absorbable / Sterile
Atóxica / No Absorbible / Estéril
CERA (Pasta de Beck) PARA HUESOS

STERILE EO

LOT XXXXXXXX

Atramat
Innovate in medical devices

01075001421002151010909396617141215

Mfg: xxxxxxxxxxxx Exp: xxxxxxxxxxxx

Version 1.4 | 2022

This Catalogue, its products and information exposed and illustrated, are solely intended for Veterinary Use and therefor exclusively for Veterinary Professionals.

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THE CONTENTS AND IMAGES SHOWN HERE ARE THE PROPERTY OF INTERNACIONAL FARMACÉUTICA, S.A. DE C.V., AND ARE DESIGNED TO BE USED EXCLUSIVELY FOR PROMOTIONAL AND DEMONSTRATIVE PURPOSES. UNDER NO CIRCUMSTANCES SHOULD THEY BE USED AS A MEDICAL REFERENCE. ALWAYS CONSULT A MEDICAL PROFESSIONAL PRIOR TO USING THIS OR ANY MEDICAL PRODUCT. INTERNACIONAL FARMACÉUTICA, S.A. DE C.V. PROVIDES THIS MATERIAL FOR THE EXCLUSIVE USE OF OUR DISTRIBUTORS AND OTHER PERSONS AUTHORIZED BY THE COMPANY IN ACCORDANCE WITH THE TERMS AND CONDITIONS ESTABLISHED BY THE COMPANY. INTERNACIONAL FARMACÉUTICA, S.A. DE C.V. CANNOT ACCEPT LIABILITY FOR DAMAGES CAUSED BY MISUSE OF ITS PRODUCTS OR OF THIS PROMOTIONAL MATERIAL.

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© Todos los derechos reservados. All copyrights reserved. Internacional Farmacéutica S.A de C.V.